





## PRESENTS CRYOLIPOLYSIS TREATMENT FOR LOCALIZED ADIPOSITY TREATMENT

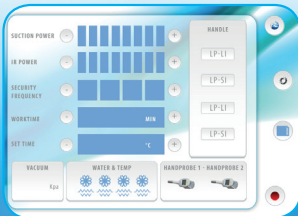

Intuitive touch screen with preset protocols and customizable programs.

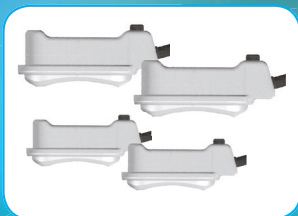

4 Handpieces to treat effectively all body's areas. Very quick cleaning of the applicators thanks to a special filter inside.



The special coating of the interior of the hanpieces with peltier cells guarantee a rapid and homogenous discent of temperature over the area trateted. The handpieces are also equipped with a temperature gauge that allows to monitor in the display the status of tissue.



The special membrane covered with a specific anti-frost serum makes the treatment safe for the tissue avoiding cold burns.




# YETI CRYOLIPOLISI


MADE IN ITALY




It is possible to reshape male and female's body.

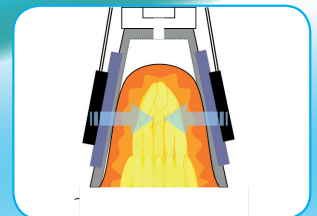

The handpiece with adjustable vacuum pressure create a massage on tissue during the treatment to avoid cellular necrosis.

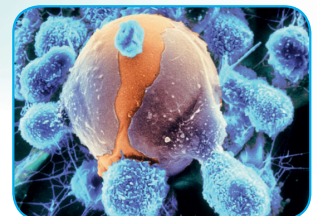

The biological effect generated on the tissue is called apoptosis: in 30 days the adipocytes will automatically ejected by macrophages.

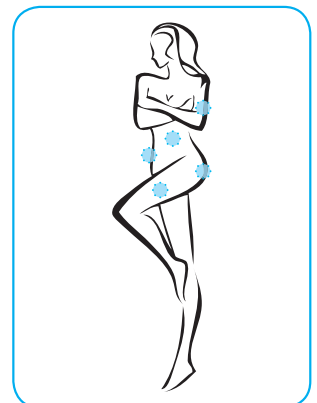

Treatable areas:

- Arms • Abdomen • Gluteus
- Thighs • Hips • Back

## THE TECHNOLOGY :

ITS Group srl R&D laboratories develop YETI, the Cryolipolisi to remove localized fat and reshape the body. Cryo= Cold, Lipo= adiposity, Lisis = elimination.


The technology born thanks to studies on Canada Amazons.

The rapid loss of cm on riders intrigued the experts who have subsequently shown that prolonged exposure to low temperatures generates apoptosis on the fat. The cellular apoptosis is a programmed death of the cell. It's a natural system of human body to eliminate the cells no longer useful to the body.

The cells that die by cellular apoptosis send a chemical signal to the immune system which activates macrophages to eliminate them.



The special applicators with vacuum make possible a uniform cooling of the tissue sucked thanks to the presence of peltier cells of on the entire inner surface. The handpiece is equipped with a temperature gauge that allow the monitoring of the tissue sucked status directly on the display. During the session, it is possible to activate a massage generated by pressures and decompressions inside the applicator that stimulate microcirculation and so avoid the risk of cellular necrosis (cellular death by unnatural causes: no signal is sent to the immune system).

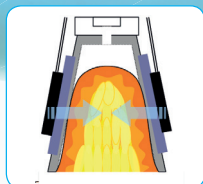

The special membranes covered with gel antifreeze avoid the risk of superficial burns by cold making the treatment safe and pleasant. In each session it is possible to lose about 20% of fat cells in the treated area. The first results are visible from 15-20 days after treatment.

Thanks to YETI in a short time you can reshape the contours of the body, remove love handles, culottes de Chevalle, abdomen, arms and thighs in a quick, painless and relaxing way!



# YETI CRYOLIPOLISI

## CONTRAINDICATIONS:



- Hepatic or renal impairment
- Raynaud's disease
- Intense cold urticaria
- Cryoglobulinemia (antibodies that become solid at lower temperatures)
- Cold fiber hypoproteinemia
- Severe diabetes
- Intolerance to cold
- Carcinoma
- Pregnancy and lactation
- Dermatitis, eczema, pimples
- Injuries
- Varicose veins, phlebitis and thrombophlebitis.
- Cancer, AIDS and hepatitis

## INFORMATIONS:



- Number of sessions: 1/5
- Maximum session interval: 30/45 days
- Session duration: 50/90

## TECHNICAL SPECIFICATIONS:

Model:	YETI
Class:	I
Type:	B
Supply:	220V 50Hz
Power:	1300W
Temperature:	max -5° C
Vacuum Pressure:	-0,02MPa
Vacuum Frequency:	4 levels
Size of big handpiece (2)	165 x 81 mm2
Size of small handpiece(2)	128 x 57 mm2
Weight:	35 Kg
Size:	L 530 x P 490 x H 1290 mm
Display:1	10 INCH, Colour touch screen 12,1 LCD
Operating Conditions:	Wetness environment 30 - 75%, Pres. 700 - 1060 hPa
Storage and transport:	Temperatur environment: -10°C < T < 70°C . Wetness environment: MAX 95%