




LIFTING AND FIRMING TREATMENT WITH MONOPOLAR, BIPOLAR AND FRACTIONAL RADIOFREQUENCY

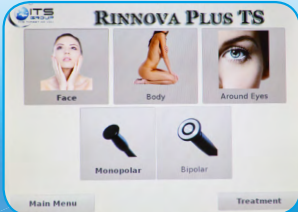


Intuitive touch screen with pre-set and customizable protocols.
Available in all languages.



Facial treatment:
• Lifting • Anti-wrinkle • Firming



Body treatment:
• Draining • Firming
• Reducing • Remodelling

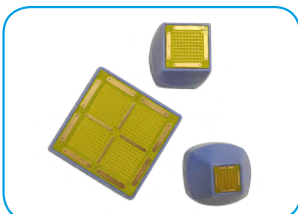

FRACTIONAL TECHNOLOGY with practical and ergonomic handpiece.

The special radiofrequency handpiece head intermittently produces 6 million energy discharges per second which generate an endogenous heat around 60°.



RINNOVA PLUS FRAX

ACTIONS:

• LIFTING • FIRMING • SKIN RESURFACING



MONOPOLAR TECHNOLOGY to treat in depth and to ensure long-lasting effects with two different frequencies to treat the face and body.



8 different sizes of MONOPOLAR ELECTRODES to easily treat all areas of the body.



Handpiece with cooled head to cool the epidermis.



BIPOLAR TECHNOLOGY for an immediate lifting effect

MADE IN ITALY

RINNOVA PLUS FRAX

THE TECHNOLOGY:

From the research of ITS Group, RINNOVA PLUS TS was created, the most efficient, safe and complete Radiofrequency technology to treat the skin laxity. Lightweight and easy to carry.


The device is equipped with a touch screen and software with pre-set and customizable protocols suitable for every type of tissue. This deep and uniform heating action causes an immediate shortening of the collagen fibres (about 1/3 of their length), simultaneously stimulating the fibroblasts to synthesize new collagen that makes the tissue taut and compact.

Rinnova combines monopolar technology, longer lasting and deeper, and bipolar technology, more superficial but immediate, and fractional technology. The fractional RF uses very intense heat (around 59/63°) with pauses of a few milliseconds that allows you to feel the heat as a pleasant warmth and avoid burns.

The handpiece head intermittently produces 6 million energy discharges per second on the tissue at a depth of 2.6-3 mm.

The result will be a lifting effect that is visible immediately after the first session that will improve and become more and more durable in the next sessions.

Tauter, more compact face, with lifted cheekbones, filled wrinkles, redesigned shape, to look at least 5/6 years younger. Also ideal for firming every area of the body, from the classic sagging of the arms to the abdomen, inner thighs and buttocks that will be lifted, reshaped, drained and toned.

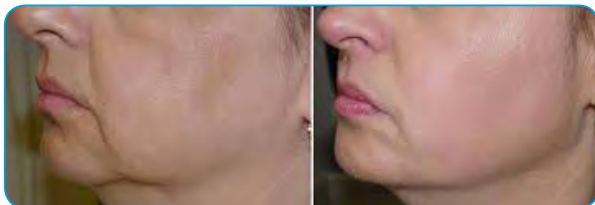

Progressive rejuvenation



Treatable areas:

- Face • Neck • Neckline • Double chin
- Arms • Abdomen • Buttocks
- Thighs • Hips • Back

CONTRAINDICATION:

- Silicone systems
- Metal implantology and protheses
- Pace Maker and general electronic devices
- Pregnancy



INFORMATION:



MONOPOLAR

- Number of sessions: 6-12
- Interval between sessions: weekly
- Duration of session: 1 h

FRAX

- Number of sessions: 2-5
- Interval between sessions: monthly
- Duration of session: 20 min

TECNICAL CHARACTERISTICS:

Model:	RINNOVA PLUS FRAX
Max Power	200W
Consumption:	
Tecnology:	Monopolar, bipolar, fractionated
Frequency:	500 - 800 kHz
Electrode sizes:	fractionated: 12x12mm, 20x20mm, 45x45mm; bipolar Ø 30mm; monopolar Ø 25, 35, 50, 70mm
Power supply:	Max 90W
Classificazion IEC 60601-1	Class I type BF
Classification 93/42/CEE	II a
Display:	Touch Screen
Compliance and Standards:	Temperature environment: 15°C - 35°C, moisture 30% - 60%, pressure 86kPa - 106kPa
Storage and transport:	Temperature environment: 5°C - 45°C, moisture 30% - 70%, pressure 86kPa - 106kPa



Via Del Cerchio 1 - 40012 Calderara di Reno (BO) - Italy
Tel. (039) 051 6467153 - Fax: (039) 051 0822612
info@itsgroupsrl.com - www.itsgroupsrl.com



ISO 9001
ISO 13485



Rev. 16.02